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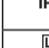
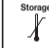
Intended use

The basal thermometer is a special digital thermometer designed to measure the basal temperature in women of childbearing age. The maximum temperature measured with the sensor is saved until the next measurement. Transferring the measurements to the Ovy app enables fertile and non-fertile days in the cycle to be predicted, as well as periods. The thermometer is not a contraceptive.

Signs and symbols

The following symbols are used in these instructions for use on the packaging and on the type plate for the device and the accessories:

	Attention
	Observe the instructions for use
	Application part, type BF
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Dispose of packaging in an environmentally friendly manner.
	Manufacturer
	Protected against foreign objects equal to or greater than 12.5 mm and against temporary immersion in water
	Batch designation
	Permissible storage and transport temperature and humidity
	Permissible operating temperature and humidity
	The CE labelling certifies that the essential provisions with the essential requirements of Directive 93/42/EEC on medical devices.

beurer OT 20	 0483 Hans Dinslage GmbH, Riedlinger Straße 28, 88524 Uttenweiler, GERMANY
FRANÇAIS	

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- mometers – Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device and the European Standard EN 60601-1-2 (In accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3 and IEC 61000-4-8), and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this device.
- Notes on electromagnetic compatibility
 - The device is suitable for use in all environments listed in the instructions for use, including domestic environments.
 - The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
 - Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
 - The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
 - Failure to comply with the above can impair the performance of the device.

⚠ Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in the instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

The basal thermometer

- measuring your basal temperature
 - Take your temperature at the same time, as soon as you wake up and before you get out of bed. You should have slept for at least 4 hours before the measurement. If you have slept for less than 4 hours before the measurement, enter the amount of sleep you have had in the Ovy app or activate „Disrupted sleep“. Do not eat anything before the measurement and avoid physical exertion.

- You can make oral, rectal or vaginal measurements, but you should always take the measurements in the same place in a cycle. We recommend taking the measurement under your tongue with your mouth closed. The thermometer should not be changed during the cycle.

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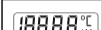
If you change the thermometer during the cycle, you must also note it as a disruption, just like a change to the place of measurement. For this cycle, a reliable evaluation is then no longer guaranteed.

	Press the On/Off button to switch on the device. A short acoustic signal will announce that the thermometer is switched on. The thermometer will carry out a self-test for about 2 seconds. During the selftest all segments of the display will appear.
	Then the reference reading 36.50 °C appears. Then the measurement symbol “°C” flashes and “Lo” is shown on the display. The thermometer is now ready for use. Insert the tip of the measuring sensor into one of the two heat pockets under your tongue, to the left or right of the roof of your tongue. The measurement sensor must have good contact with tissue.
	

Close your mouth and breath calmly through your nose so that the air from your breathing does not influence the measurement. During the measurement, the current temperature is displayed continuously, and the “°C” symbol flashes on and off. The measurement is completed when a stable temperature has been reached. When this stage has been reached, an acoustic signal is heard, the “°C” symbol stops flashing and the temperature measured is displayed. We recommend a measuring time of 3 minutes. If the temperature stability is reached earlier, the acoustic signal will announce that the measurement is complete. Please wait for the end of the measurement before you remove the thermometer from the place of measurement. The measured value is saved until the next measurement. The device switches off automatically approx. 10 minutes after the end of the measurement. You can also switch the thermometer before this time by pressing the On/Off button.

Close your mouth and breath calmly through your nose so that the air from your breathing does not influence the measurement. During the measurement, the current temperature is displayed continuously, and the “°C” symbol flashes on and off. The measurement is completed when a stable temperature has been reached. When this stage has been reached, an acoustic signal is heard, the “°C” symbol stops flashing and the temperature measured is displayed. We recommend a measuring time of 3 minutes. If the temperature stability is reached earlier, the acoustic signal will announce that the measurement is complete. Please wait for the end of the measurement before you remove the thermometer from the place of measurement. The measured value is saved until the next measurement. The device switches off automatically approx. 10 minutes after the end of the measurement. You can also switch the thermometer before this time by pressing the On/Off button.

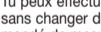
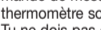
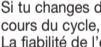
Memory

The measured value is automatically saved. To display this value, press the On/Off button for more than 2 seconds. The last measured value is then shown. Once the button is released, the reference value of 36.5 °C and “Lo” are shown on the display. The thermometer is ready to take a measurement and the previously stored measured value is automatically replaced by the new measured value.

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Replacing the battery

- Replace the battery when the battery status indicator appears on the display.
- Slide back the battery compartment cover to remove it.
- Pull the battery holder carefully out of the housing by approx. 1 cm.
- Use a non-metallic object to push the battery out of the housing.
- Insert a new battery (type LR41 1.5V) with the + symbol facing upwards.
- Push the battery holder back into the housing and reattach the battery compartment cover. Ensure that you do not damage or displace the seal.

Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

Do not dispose of batteries containing harmful substances:
Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.
This battery contains: Pb, Hg.

⚠ Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical advice.
- Avoid contact with hazardous! Small children may swallow and choke on batteries or the battery compartment. Store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and wash the affected area with water until completely clean.
- Protect batteries from excessive heat.
- Risk of explosion! Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, store the batteries out of the battery compartment.
- Use different or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

Cleaning the device

Clean the basal thermometer before and after each use with a soft cloth and isopropyl alcohol diluted with water or cold soapy water. The device can be immersed in water or a disinfectant solution to clean it. The maximum

immersion time is 30 minutes. Do not sterilise the device by boiling, with gas or with an autoclave.

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