DE Massage-Fußwärmer Gebrauchsanweisung	2	RU Грелка для ног Инструкция по применению	47
EN Massaging foot warmer Instructions for use	10	PL But rozgrzewający z funkcją masażu Instrukcja obsługi	55
FR Chauffe-pieds massant Mode d'emploi	17	DA Massage-fodvarmer Betjeningsvejledning.....	63
ES Calientapiés con masaje Instrucciones de uso	25	SV Massagefotvärmare Bruksanvisning	70
IT Scaldapiedi con massaggio Istruzioni per l'uso	33	NO Massasjefotvarmer Bruksanvisning	77
TR Masajlı ayak ısıtıcısı Kullanma Talimatı	40	FI Hierova jalkalämmitin Käyttöohje	84

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Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, air and baby.

With kind regards
Your Beurer team


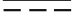
Contents

<p>1. About the Shiatsu massage 11</p> <p>2. Included in delivery..... 12</p> <p style="padding-left: 20px;">2.1 Unit description 12</p> <p>3. Important instructions</p> <p style="padding-left: 20px;">Retain for future reference..... 12</p> <p>4. Intended use..... 14</p> <p>5. Operation..... 14</p> <p style="padding-left: 20px;">5.1 Initial use..... 14</p> <p style="padding-left: 20px;">5.2 Switching on/off the Shiatsu function (massage) 14</p>	<p>5.3 Switching on/off the heat function (☹☹☹).. 14</p> <p>5.4 Automatic switch-off..... 14</p> <p>6. Cleaning and maintenance..... 14</p> <p>7. Storage..... 15</p> <p>8. Disposal..... 15</p> <p>9. What if there are problems? 15</p> <p>10. Technical data 15</p> <p>11. Warranty / service..... 16</p>
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

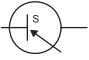Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Read the instructions!		Do not wash
	Do not insert pins!		Do not bleach
			Do not dry in a tumble dryer
	Do not use folded or rucked!		Do not iron
	Not to be used by very young children (0 - 3 years).		Do not dry-clean
	Only use in dry rooms	IPX1	Drip proof
	Dispose of packaging in an environmentally friendly manner		Manufacturer

	This product satisfies the requirements of the applicable European and national directives.		The products demonstrably meet the requirements of the technical Regulations of the EAEU.
	The device has double protective insulation and therefore complies with protection class 2.		Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
	Switching mains part		Energy efficiency level 6
IP20	Protected against solid foreign objects 12.5 mm in diameter and larger		Direct current The device is suitable for use with direct current only
	Short-circuit-proof safety isolating transformer		Polarity
	The textiles used for this device meet the stringent human ecological requirements of Oeko Tex Standard 100, as verified by Hohenstein Research Institute.		
	WARNING: Warning of risks of injury or health hazards		
	CAUTION: Safety information about possible damage to appliance/accessories.		
	NOTE: Important information.		

1. About the Shiatsu massage

Shiatsu is a type of body massage that developed in Japan over 100 years ago, based on ideas taken from traditional Chinese medicine. The basic principle is that there is a system of energy channels (meridian system) within the human body.

The objective of a Shiatsu massage is to boost the physical, emotional and mental well-being of the treated person. To this end, energy blockages in the energy channels must be removed and the self-regulating powers of the body must be stimulated.

Shiatsu therapy achieves this through the application of pressure in flowing movements along the energy channels (meridians). Although Shiatsu literally translates as 'finger pressure', treatment is not just given with

the fingers but also with the heels of the hand, elbows and knees. Therapists use their whole body weight to create the necessary pressure.

Massaging a specific point on the sole of the foot can influence the corresponding organs of the body. However, we recommend studying relevant literature and asking your doctor before using point reflex zone massage to achieve the desired effect for you.


2. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 Shiatsu foot warmer with lining
- 1 Mains part
- 1 Instructions for use

2.1 Unit description

- 1. Mains part
- 2. Cable
- 3. Connector socket
- 4. Massage button
- 5. Heat button
- 6. Shiatsu foot warmer
- 7. Lining

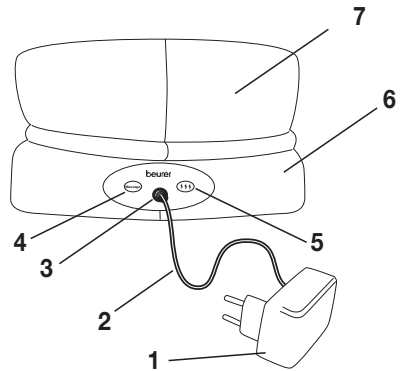3. Important instructions

Retain for future reference

WARNING

- Non-observance of the following notes may result in personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information is not only intended to protect your health and the health of others, it should also protect the product. For this reason, pay attention to these safety notes and include these instructions when handing over the product to others.
- This foot warmer must not be used by people who are not sensitive to heat or by vulnerable people who may not be able to react to overheating (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).
- This foot warmer must not be used by very young children (0-3 years old) as they are unable to respond to overheating.
- This foot warmer can be used by children older than 3 and younger than 8 years of age provided they are supervised. For this, the controller must always be set to the minimum temperature.

- This foot warmer may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised and have been instructed on how to use the foot warmer safely, and are fully aware of the consequent risks of use.
- Children must not play with the foot warmer.
- Cleaning and user maintenance must not be performed by children unless supervised.
- This foot warmer is not designed for use in hospitals.
- The device is only intended for domestic/private use, not for commercial use.
- This foot warmer must not be used to keep animals warm.
- Do not use this foot warmer with shoes.
- Do not insert needles.
- Only use in dry rooms.
- Do not use if wet.
- Connect the unit only to the mains voltage listed on the type plate.
- Use only the original mains part provided (see “Technical data” chapter).
- The electrical and magnetic fields emitted by this foot warmer may interfere with the function of a pacemaker. However, they are still well within the limits: electrical field strength: max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 millitesla. Please consult your doctor and the manufacturer of your pacemaker before using this foot warmer.
- Do not pull, twist or make sharp bends in the cables.
- If the cable and controller of the foot warmer are not positioned properly, there may be a risk of becoming entangled in, being strangled by, tripping over, or stepping on the cable and controller. The user must ensure that excess lengths of cable, and cables in general, are safely routed.
- Please check this foot warmer frequently for signs of wear and tear or damage. If any such signs are evident, if the foot warmer has been used incorrectly or if it no longer heats up, it must be checked by the manufacturer before being switched on again.
- Under no circumstances should you open or repair the foot warmer (including the accessories) yourself because faultless functionality can no longer be guaranteed thereafter. Failure to observe this will invalidate the warranty.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

- It is essential to observe the information relating to the following chapters: Operation, Cleaning and maintenance, Storage.
- If you should have any questions about using our devices, please contact Customer Services.

4. Intended use

This product is intended for warming and massaging human feet.

WARNING

Do not use the foot warmer with shoes, as they reduce your perception of heat and dirty the foot warmer. This foot warmer is not designed for use in hospitals or commercial use. The following groups in particular must not use the device: infants, small children, vulnerable persons, persons with low sensitivity to heat or animals. Otherwise, there is a risk of injury.

5. Operation

5.1 Initial use

- Connect the mains part cable to the connector socket on the back of the foot warmer.
- Insert the mains part into the wall socket.

5.2 Switching on/off the Shiatsu function (massage)

- To switch on the Shiatsu function, press the massage button on the back of the foot warmer. The massage heads will begin to rotate.
- To switch off the Shiatsu function, press the massage button on the back of the foot warmer again. The massage heads will stop.

IMPORTANT

If too much pressure is exerted on the massage heads, the massage function will remain off for safety reasons. In order to use the foot warmer again, briefly remove the cable from the connector socket and then reinsert it.

5.3 Switching on/off the heat function (☺☺☺)

- To switch on the heat function, press the heat button (☺☺☺) on the back of the foot warmer. A red LED above the heat button will light up.
- To switch off the heat function, press the heat button on the back of the foot warmer again. The red LED above the heat button will go out.

5.4 Automatic switch-off

The foot warmer will switch off after approx. 30 minutes for safety reasons. Press the massage or heat button again if you wish to continue using the foot warmer.

Switch the foot warmer off and remove the mains part from the socket if you do not wish to continue using the foot warmer after the timed switch-off.

6. Cleaning and maintenance

IMPORTANT

Before cleaning, always remove the mains part from the socket first. Otherwise, there is a risk of damage.

Small marks on the foot warmer can be removed with a damp cloth and, if necessary, with a little liquid detergent for delicate laundry.

IMPORTANT

Please note that the foot warmer must not be washed, chemically cleaned, wrung out, tumble dried, put through a mangle or ironed. Failure to observe this may result in damage to the foot warmer.

- The removable lining is machine washable at 30°.
- Use a delicate laundry detergent and measure it out according to the manufacturer's instructions.

IMPORTANT

Only use the foot warmer again once the foot warmer and lining have fully dried. Otherwise, there is a risk of damage.

7. Storage

If you do not plan to use the foot warmer for a long period of time, we recommend that you store it in the original packaging.

IMPORTANT

- Please allow the foot warmer to cool down before storing. Failure to observe this may result in damage to the foot warmer.
- To avoid damaging the foot warmer, do not place any objects on top of it whilst storing.

8. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

9. What if there are problems?

Problem	Cause	Solution
The foot warmer does not warm up or the massage function cannot be activated while: <ul style="list-style-type: none"> – the mains part is connected to a working socket. – the mains part cable is connected to the connector socket on the foot warmer. – both functions (heat + massage) are switched on. 	The foot warmer is faulty.	Send the foot warmer for servicing.

10. Technical data

For technical data about the foot warmer, see the type plate label on the foot warmer.

For technical data about the mains part, see the type plate label on the mains part.

Permissible mains parts:

EU mains plug: IVP1200-2000G

UK mains plug: IVP1200-2000B

Model identifier	IVP1200-2000B
Input voltage	100 V - 240 V
Input frequency	50/60 Hz
Output voltage	12.0 V

Output current	2.0 A
Output power	24.0 W
Average efficiency during operation	86.3%
Efficiency load	82.0%
load power consumption	0.09 W

11. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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